# Operator Manual

## OUT-SIDER®

ASL-275

02-03-2022

P/N 275-O

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**Not compatible for use with RVs  
(motorhomes, buses, fifth wheels or travel trailers)**

[www.bruno.com](http://www.bruno.com)

# PRODUCT REGISTRATION FORM

Please register your Bruno lift in one of two ways:

- Fill it out online at [www.bruno.com/forms/product-registration](http://www.bruno.com/forms/product-registration)
- Complete the enclosed Product Registration card and return via mail

## Serial Number:

Please note a serial number is found on your Bruno lift. The serial and model number will be needed for your dealer to file a warranty claim, request parts or service the unit.

## Authorized Bruno Dealer:

Bruno products must be installed and maintained by an authorized Bruno dealer, or the warranty will be null and void.

## TABLE OF CONTENTS

Safety . . . . .	3
Operation	
Scooters and Powerchairs . . . . .	4
Powerchairs Using Hold-Tite . . . . .	5
Using Retractable Belts . . . . .	6-7
Swing-Away . . . . .	8
<b>Emergency Operation</b> . . . . .	9
Maintenance . . . . .	9
Troubleshooting . . . . .	10
Warranty . . . . .	11

## SAFETY

### **WARNING**

***Observe all safety precautions***


**Failure to do so could result in damage to your vehicle or property as well as personal injury.**

- Never exceed the load capacity of the lift **installed according to Bruno instructions:** 350 lb (159 kg)
- Never use the ASL-275 to lift or transport people.
- Do not allow children to play on or under the lift platform.
- Regularly examine the ASL-275 for loose hardware and attachments. Tighten as necessary.
- **Load and unload from/to as level an area as possible.**
- Maintain a safe distance from moving lift parts.
- Avoid unloading into vehicular traffic.
- Make sure mobility device (scooter or powerchair) is OFF before attaching it to the lift.
- Do not remove any guards or covers.
- Do not operate the ASL-275 while seated on the mobility device.
- Operate the ASL-275 from the side of the unit, not the front.
- Keep feet and legs at a safe distance from the lift platform.
- Check for obstructions under the platform prior to lowering the platform.
- Adjust your driving to address the handling limitations imposed on the vehicle by the additional weight of the ASL-275 plus the mobility device.
- Reduce your speed until you become familiar with your vehicle's turning and braking performance with the lift installed.
- Avoid sudden maneuvers. Anticipate turns, stops and lane changes.
- Be aware that the addition of the ASL-275 and a mobility device may cause your vehicle to be more susceptible to crosswinds.
- Use caution when driving a lift-equipped vehicle on slippery road surfaces (snow, ice, loose gravel).
- Reduce your speed when travelling on bumpy roads, over railroad crossings, street gutters and other road surface irregularities.
- Be aware that the ASL-275 adds length to your vehicle. Back up carefully.
- Remove any loose items from the mobility device before transporting it. Be sure batteries are securely strapped or clamped in place.
- Before taking the lift through a car wash:
  - verify that the car wash is a touchless system;
  - remove the mobility device from the lift;
  - run platform UP to folded position.

# OPERATION – SCOOTERS AND POWERCHAIRS

## **General Instructions**

1. Open the keyswitch cover.
2. Flip open switch cover on mast tube.
3. Insert the key and turn it to ON position.
4. Press and hold the DOWN switch to lower the platform.
5. Release the switch when the platform reaches the ground.
6. Position the mobility device on the platform.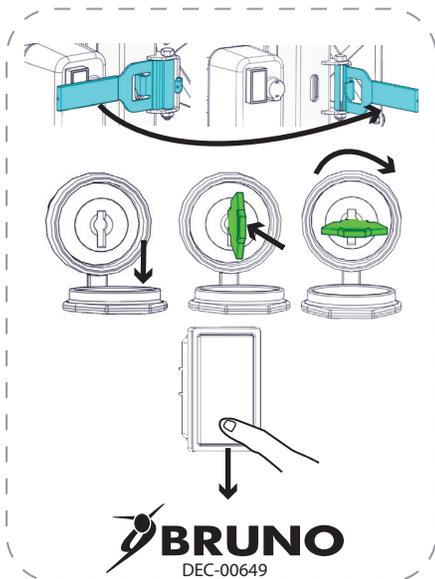


## **Scooters Using Hold-Tite Mechanism**

### **Loading**

1. Flip open switch cover.
2. Press and hold the UP switch to raise the platform/mobility device, and to automatically lower the Hold-Tite mechanism.
3. Release the switch when the platform reaches the uppermost position.
4. Verify that the Hold-Tite foot contacts the scooter deck and secures the scooter. The scooter should not move. Adjust the Hold-Tite foot, if necessary (a) and (b).

### **Unloading**

1. Open the keyswitch cover.
2. Insert the key and turn to the ON position.
3. Flip open the switch cover.
4. Press and hold the DOWN switch to lower the platform/mobility device, and to automatically lift the Hold-Tite mechanism.
5. Release the switch when the platform reaches the ground.
6. Drive it off of the platform.
7. Press and hold the UP switch to raise the platform and fold it in its raised home position.
8. Turn the key to the OFF position.
9. Close the keyswitch cover.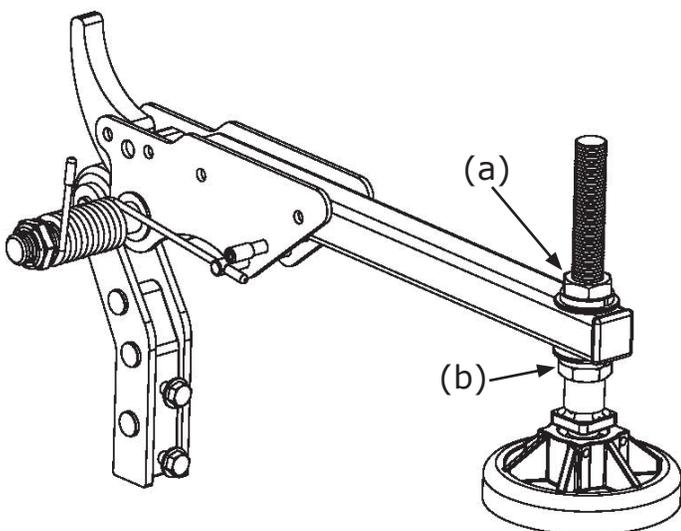

# POWERCHAIRS USING HOLD-TITE

---

## ***Powerchairs Using Hold-Tite Mechanism:***

### ***Loading***

1. Raise the powerchair armrests.
2. Pull the T-handle out and turn it to the horizontal position. Release.
3. Rotate the pivot arm over the powerchair seat (approx. 90 degrees).
4. Turn the T-handle back to the vertical position. Release. It will click into the locked position.
5. Flip open the switch cover.
6. Press and hold the UP switch to raise the platform/mobility device.

*NOTE: The lift will make a ratcheting sound when it has reached the upper stop position.*

7. Release the switch when the pivot arm firmly compresses the powerchair seat cushion. (It should be difficult to insert your fingers under the pivot arm at the edge of the cushion. See photo at left.)


### ***Unloading***

1. Raise the powerchair armrests.
2. Flip open the switch cover.
3. Press and hold the DOWN switch to lower the platform/mobility device.
4. Pull the T-handle out and turn it to the horizontal position. Release.
5. Rotate the pivot arm off the powerchair seat (approx. 90 degrees).
6. Turn the T-handle back to the vertical position. Release. It will click into the locked position.
7. Drive the powerchair off the platform.
8. Press and hold the UP switch to raise the platform and fold it in its home position.
9. Turn the key to the OFF position.
10. Close the keyswitch cover.

# USING RETRACTABLE BELTS

## Scooters, Powerchairs Using Retractable Belts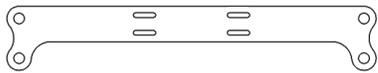


Retractable belt hooks attach to this docking plate.

Have an authorized Bruno dealer remove the mobility device seat, mount the docking plate (see left) between the seat and the swivel plate of the mobility device and place the seat assembly back on the mobility device.

### **Loading**

1. Open the keyswitch cover.
2. Insert the key and turn it to the ON position.
3. Make sure an authorized Bruno dealer has installed the docking plate shown earlier in this manual.
4. Flip open the switch cover.
5. Press and hold the UP switch to raise the platform/mobility device.
6. Release the switch when the platform reaches the uppermost position.
7. Secure the mobility device to the platform using the ratcheting belts.
8. Turn the key to the OFF position.
9. Remove the key from the key switch.
10. Close the key switch cover.

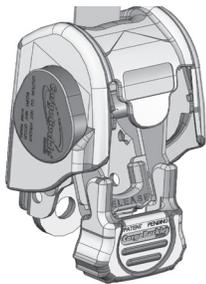## Retractable Belt Operation

Fig. 1

Fig. 2

Fig. 3

Fig. 4

### **Platform belts**

- Press inner release tab down (Fig. 2).
- Push large orange tab UP (Figs. 3 and 4).
- Pull strap up and hook to mobility device.
- Press inner release tab and hold while pressing large orange tab down to home position (Fig. 1).

*NOTE: You will hear the tab click into its home position. The strap will "auto ratchet" to secure the mobility device on the ASL-275.*

# USING RETRACTABLE BELTS

---

## ***Scooters, Powerchairs Using Retractable Belts:***

### ***Unloading***

1. Loosen the belts pressing the inner release tab.
2. Unhook the retractable belts from the plate secured to the bottom of the mobility device seat.
3. Open the keyswitch cover.
4. Insert the key and turn it to the ON position.
5. Flip open the switch cover.
6. Press and hold the DOWN switch to lower the platform/mobility device.
7. Release the switch when the platform reaches the ground.
8. Press and hold the UP switch to raise the platform and fold it in its home position.
9. Turn the key to the OFF position.
10. Close the keyswitch cover.

# SWING-AWAY

**! WARNING**

Risk of personal injury and/or property damage

Do not operate Swing-Away with a mobility device on board

**! WARNING**

**PINCH POINTS!**



- lever linkage
- lock pin-receiver

DEC-00528

**! WARNING**

Risk of personal injury and/or property damage

Load and unload from and to as level an area as possible.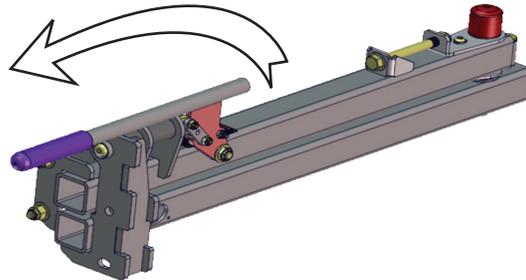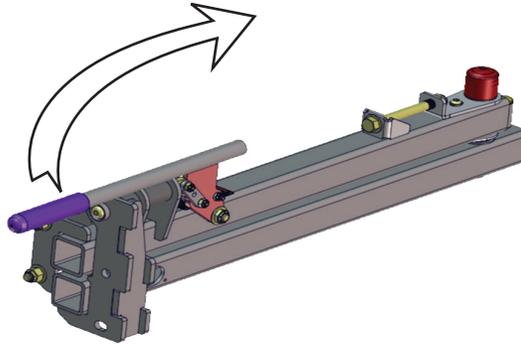

The Swing-Away allows access to the vehicle rear hatch when an ASL-275 is installed on the vehicle.

***Do NOT operate the Swing-Away with a mobility device on board.***

To gain access to the vehicle's hatch:

1. Lift the latch handle.
2. Swing the ASL-275 out and away from the vehicle.
3. The vehicle hatch may now be opened.

To return the Swing-Away to its closed and locked position:



1. Close the vehicle hatch.
2. Swing the ASL-275 in toward the vehicle.
3. Push the lever back to the closed position.

## EMERGENCY OPERATION

***Have an able-bodied person operate the emergency crank.***

To perform manual operation of lift:

1. Rotate and lift cap covering tail shaft in order to access motor shaft.
2. Turn the key switch to OFF.
2. Slide the emergency crank on the end of the motor shaft as shown.
3. Rotate the crank:
  - ***clockwise to raise*** platform;
  - ***counterclockwise to lower*** platform.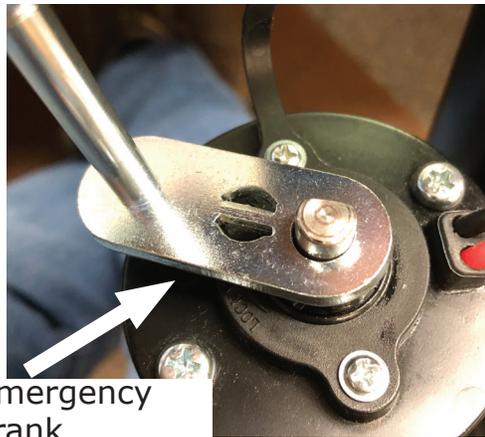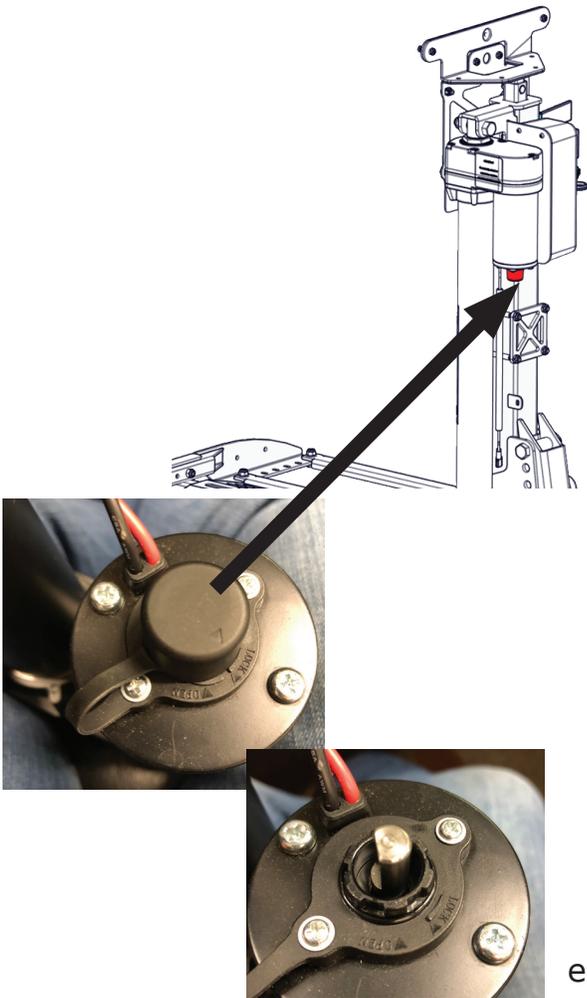

emergency crank

### **NOTE:**

A 3/8" electric drill may be used to crank shaft during emergency operation.

## MAINTENANCE

Bruno recommends having a qualified Bruno dealer clean, inspect and perform any necessary repairs and adjustments at least every twelve (12) months.

Maintenance is recommended more often if the lift is used (a) many times per day and/or (b) in harsh environments.

***WARNING: Read vehicle manufacturers (OEM) instructions about the vehicle's electrical system BEFORE working on the vehicle. Vehicle OEMs have model specific requirements about battery disconnection when performing work on a vehicle. Disconnecting batteries incorrectly or working on a vehicle with batteries improperly in the circuit MAY cause loss of vehicle data, damage vehicle wiring system or even cause airbags to deploy. This information can be found in the VEHICLE OWNER'S MANUAL or the VEHICLE REPAIR GUIDE.***

## TROUBLESHOOTING

<b><i>PROBLEM</i></b>	<b><i>PROBABLE CAUSE</i></b>	<b><i>REMEDY</i></b>
<b>Lift does not operate in either direction</b>	<ul style="list-style-type: none"> <li>• keyswitch not turned to ON position</li> </ul>	<ul style="list-style-type: none"> <li>• Turn keyswitch to ON position.</li> </ul>
<b>Lift operates slowly, lacks power or stalls</b>	<ul style="list-style-type: none"> <li>• weak vehicle battery</li> </ul>	<ul style="list-style-type: none"> <li>• Have battery charged or run engine at fast idle for several minutes before attempting to operate lift.</li> </ul>
<b>Lift platform does not fold (no mobility device on platform)</b>	<ul style="list-style-type: none"> <li>• obstruction in lift mechanism</li> <li>• vehicle hitch too low (too close to ground)</li> </ul>	<ul style="list-style-type: none"> <li>• Have the lift inspected and repaired by an authorized Bruno dealer. Drive with the lift in the unfolded, down position. (It is OK to drive unfolded, with no mobility device on the platform).</li> </ul>

---

## **BRUNO GOLD LIMITED WARRANTY FOR VEHICLE LIFT 3 Years**

Bruno Independent Living Aids, Inc.© (Bruno) is so confident in our USA designed and built units that we provide every original buyer with the following "Industry Leading" warranty for your peace of mind.

The quality of parts we use and the care our employees take allow Bruno to provide the original buyer of its Bruno Vehicle Lift with a **3-year warranty** that all parts are free of defects in material and workmanship.

Bruno's warranty periods begin with **the date of the installation** of the Bruno Vehicle Lift.

Bruno Authorized Dealers are trained to evaluate your unique situation and then install the best Bruno product to meet your needs. The trained installation and service technicians employed by Bruno Authorized Dealers have the appropriate equipment and resources needed to correctly install and maintain your new vehicle lift and are the first contact for any warranty or service needs.

### **SPECIFICS OF THE WARRANTY COVERAGE:**

- ◇ All Parts are free of defects in material and workmanship for a period of three years after the date of installation.
- ◇ All labor to address any product defect by a Bruno Authorized Dealer is covered for the first 30 days after installation.

### **WHAT IS NOT WARRANTY COVERAGE:**

- ◇ Damage to units caused by misuse, abuse, accidents, neglect or modifications NOT made by a Bruno Authorized Dealer.
- ◇ Consumable items.
- ◇ All labor by a Bruno Authorized Dealer after the first 30 days since installation.
- ◇ Reasonable wear and tear.


### **ADDITIONAL NOTES:**

All Service and Warranty issues are handled through the Bruno Authorized Dealer that installed the unit.

- ◇ In no event shall Bruno be responsible for indirect, incidental or consequential damages, whether such damages arise based on claims based on contract, warranty, tort (including negligence, strict liability or product liability).
- ◇ Bruno reserves the right to replace, repair or, if not commercially practical and/or feasible, to elect to refund the purchase price at its sole discretion.
- ◇ This warranty gives the original buyer specific legal rights, and you may also have other rights which vary from state to state. Bruno specifically does not authorize any person to extend the time or scope of this warranty.

For more detailed information regarding this limited warranty, please contact Bruno by writing:

Bruno Independent Living Aids, Inc.  
Attn: Service Department  
1780 Executive Drive  
Oconomowoc, WI 53066 USA

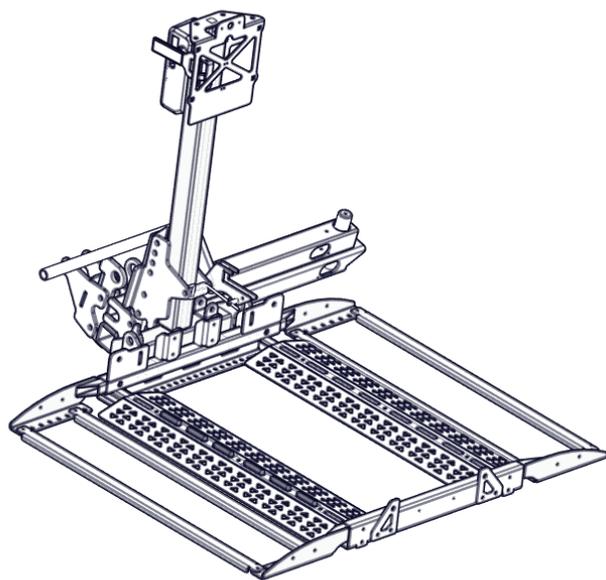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