



WHILL Model Fi: Airline Handling Guide

For proper airport personnel handling of WHILL Model Fi power chair.
Attach this document to device in a conspicuous area.

Owner's Contact Information:

Name: _____

Phone: _____

Device Serial #: _____



Battery


Type: Rechargeable Lithium-ion
Specs: 25.3 V; 269 Wh
Other: Removable from device.

Requires removal and storage in cabin for airline travel.



Freewheel Mode

Disengage brakes by pushing down the two brake levers located under the seat in front of the rear wheels. Fold back support down.

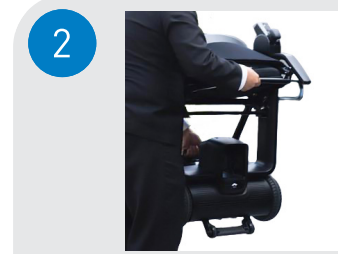

Lifting Device

To avoid damage, DO NOT lift the device by its armrests. Only lift the device by the frame tubing.



Engage Brakes

When the Model Fi reaches the cargo area, load device in upright position, then lock the brakes by lifting the two red levers up. This will prevent the device from shifting in-flight.

At destination, safely return the device to the owner at the gate by following steps 1 and 2 in reverse order.

WHILL rechargeable lithium-ion battery has been tested on UN-Manual of Tests and Criteria Part 3 Subsection 38.3 and certified to meet the applicable ICAO, IATA and FAA requirements. However, WHILL Model Fi personal mobility device has not been specifically tested for air transport.